
Instructions for use ESENTA™ Strips /
Gebrauchsanweisung – ESENTA™ Streifen /
Notice d'utilisation des renforts ESENTA™/
Istruzioni per l'uso delle strisce ESENTA™ /
Brugsvejledning til ESENTA™ kantsikring /
Instrucciones de uso de las tiras ESENTA™ /
Καyttöohjeet ESENTA™ -kiinnitysteipille /
Upute za uporabu traka ESENTA™ /
Lietošanas pamācība ESENTA™ strēmeles /
Juostelių „ESENTA™“ naudojimo instrukcija /
Instruções de utilização das tiras ESENTA™ /
Plaastriribade ESENTA™ kasutusjuhend /
Инструкция за употреба за ленти ESENTA™ /
Bruksanvisning ESENTA™-strips /
Gebruiksaanwijzing ESENTA™-strips /
Instrukcja stosowania paszków ESENTA™ /
Bruksanvisning for ESENTA™ plateforlengere /
Návod k použití pásků ESENTA™ /
Návod na použitie adhezívny pásky ESENTA™ /
Οδηγίες χρήσης για τις ταινίες ESENTA™ /
Notkunariebõbeinngar fyrir ESENTA™ strímila /
ESENTA™テープの使用方法 /
Használati utasítás – ESENTA™ rögzítőcsíkok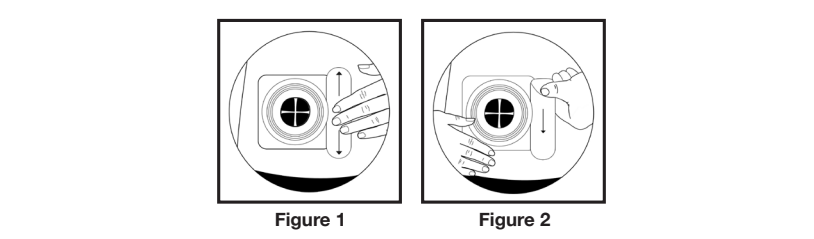

<p> ENGLISH</p>
--

General information

ESENTA™ Strips are non-sterile hydrocolloid strips, covered with a top layer of a breathable polyurethane. ESENTA™ Strips contain pectin, gelatin, sodium CMC and polyisobutylene. ESENTA™ Strips are not manufactured using natural rubber or latex.

Intended use

ESENTA™ Strips are intended to be used as additional fixations for the edges of skin barriers around the stoma.

Clinical Benefits:

- Aids adhesion of the baseplate.
- Aids in obtaining a good seal.
- Helps to increase stoma system wear time.

Indications

- ESENTA™ Strips are suitable for use on intact skin only.
- To be used with a colostomy, ileostomy or urostomy.

Contraindications

Do not use to cover broken skin or open wounds or on individuals who are sensitive to or who have developed sensitivity or allergic reaction to the product.

Patient target groups:

For individuals with an ileostomy, colostomy or urostomy.

Intended Users

Ostomates, healthcare professionals and carers.

Warnings and precautions

Should irritation occur, discontinue use, and consult your healthcare professional. This device is for single-use only and should not be reused. Re-use may lead to increased risk of infection and cross-contamination. Physical properties of the re-used device may no longer be optimal for intended use.

INSTRUCTIONS/Performance characteristics

Preparation

- Choose correct size and form of strips suitable for stoma appliance.
- The strips should overlap the skin barrier for at least 1.5 to 2cm.
- Clean and dry the skin and ensure it is free from solvent or greasy substances

Application

- Remove the release papers.
- Apply the strips to the skin overlapping the skin barrier and smooth down for 30 seconds.

- Figure 1
- Use one or more strips around baseplate as required.
- Wear as long as comfortable and secure; individual wear time may vary.

Removal

Support the skin and gently peel off the strips from the body. (Figure 2)

Disposal

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

© 2023 Convatec Inc.

®/™ All trade marks are the property of their respective owners.

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