


# SUR-FIT®/ACTIVE LIFE®

## TAIL CLOSURE /PINZA/ATTACHE

Instructions for use

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Mode d'emploi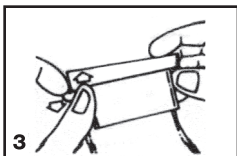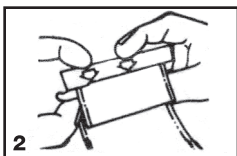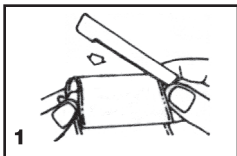

### ENGLISH - Instructions for use

**NOTE:** If the belt collar is to be used, the tail closure should be attached after the pouch has been passed through the collar opening.

1. Tail closure may be attached *before* pouch application. Fold pouch tail over "knife" edge of closure *only once* as shown. Hold in place.
2. Press firmly on raised bar of closure (especially at center) until it "snaps" securely closed.
3. To remove closure, separate "knife" edge from bar as shown.

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ConvaTec Inc.  
Greensboro, NC 27409  
USA  
1-800-422-8811

ConvaTec Limited, Deeside, CH5 2NU, UK

Imported by:  
DKSH Taiwan Ltd.  
10th Floor, No. 22, Lane 407,  
Tiding Blvd., Sec. 2,  
Neihu Technology Park, Taipei City 114,  
Taiwan, R.O.C.

Sponsored in Australia by:  
ConvaTec (Australia) Pty Ltd.  
Unipark Monash, Building 2, Ground Floor  
195 Wellington Road  
Clayton, Victoria 3168  
(P.O. Box 63, Mulgrave, Victoria, Australia 3170)

**Canada**  
**Legal Manufacturer / Fabricant légal**  
ConvaTec Inc.  
7815 National Service Road, Suite 600  
Greensboro, NC 27409 USA  
1-800-422-8811

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ARTWORK SPECIFICATION	
Artcode	1001186G2
Artwork Description	INSERT CLIP TAIL ACTIVE-LIFE US
ICC*	175635
SAP*	1000601
KOM/REF*	N/A
IS Code*	N/A
Component Type	Insert
LR#	LR-004882
Existing Art Code being Revised	1001186G1
REQUIREMENT	SPECIFICATION
Bucket	US
Local Code(s)	N/A
Size Code / Dieline	10064
Dimensions	flat: 57.2mm x 228.6mm (2 1/4" x 9") folded:57.2mm x 19.1 mm (2 1/4" x 3/4")
Color(s) / Colour(s)	Black
Specification Reference(s)	MT56-003
Bar Code: UPC (A) or GTIN**	N/A
Bar Code: PZN – Code 39**	N/A
Bar Code: JAN-13**	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
HISTORY	
06-OCT-2008 / 1001186C1: Revise Entity Information. Update UK/US Addresses. Remove Local Market Addresses. Insert color changed to black. Artcode on insert changed to 1001186C1, Registration symbol removed from ConvaTec logo. 21-JUN-2011 / 1001186C2: Removed outdated "people" graphic. Added NAI Global Contact List. Artcode updated to next revision level. 23-JUL-2013 / 1001186C2: No change to artwork or artcode. No additional proof required. Size code 10064 added to spec. 11-MAY-2015 / 1001186G1: Address change From: Skillman, NJ 08558 To: Greensboro, NC 27409, SAP art code version changed to the G1, Revision date updated (Issue date changed to Revision date if applicable), Artwork Specification updated. 23-JUN-2017 / 1001186G1: Required to change the source of manufacturing of the art components produced on the O.E. Transfer Project, that will transfer from Greensboro, NC to the Haina Dominican Republic facility. No update to art components required. No Change to Art codes required. 31-OCT-2017 / 1001186G1: Updates required to the Readability and Printability for the art Component. No additional change to the existing art component. No change to the existing art code. 09-NOV-2017 / 1001186G1: No change to art component. Art clean-up for read ability and print ability for print vendor. 17-July-2023 / 1001186G2: Legal manufacturer Canada address added. Canadian phone number removed.	